

Instruction book of Slimming Queen

Multipoles RF Vacuum Cavitation Body Slimming Machine



1. Precautions

This product must be used strictly in accordance with the instructions in this manual, and the company shall not be held responsible for all consequences caused by misuse of methods other than those in the manual.

Before use

- Be sure to read the instruction manual carefully before starting the machine. The operator must be familiar with the instruction manual and operation rules, and the instrument must be managed by a specific person;
- The setting of operating parameters should be set under the guidance of professionals;
- Confirm whether the instrument is in normal operation before use;
- To confirm whether all wires and parts are original and correctly connected, and to confirm that the grounding wire is correctly connected;
- Do not use the instrument in the bathroom or in places with high humidity

Using

- When the instrument or the patient is abnormal during the operation, the operator immediately leaves the operating handle away from the patient and presses the stop switch to stop the use of the instrument;
- During the operation of the instrument, it is necessary to continuously observe the situation of beauty guests and the instrument, and pay attention to whether there is any abnormality at any time. If there is any abnormality in the skin when using this product, please stop using it and communicate with the doctor in time;
- The instrument should be used correctly by a skilled operator, and persons other than the operator should not use the instrument;

After use

- 1) After the operation, put the control handle back on the corresponding handle hanger.
- 2) After using the operating handle, the probe should be cleaned for the next use.

Product storage

- Please keep the power off
- Do not store near microwave ovens or high-voltage instruments
- Please do not expose the product to high temperature environment above 60°C

2. Machine and parts

Machine parts



4 poles RF



6 poles RF



9 poles RF



6 poles RF + Vacuum



80K Cavitation



Cooling Hammer



Lipolaser paddle L




Lipolaser paddle S

3. Machine Installation Instructions

1) Install the transparent acrylic handle hanging plate and tighten




2) Filter installation

Plug the filter into the rightmost jack on the back of the machine

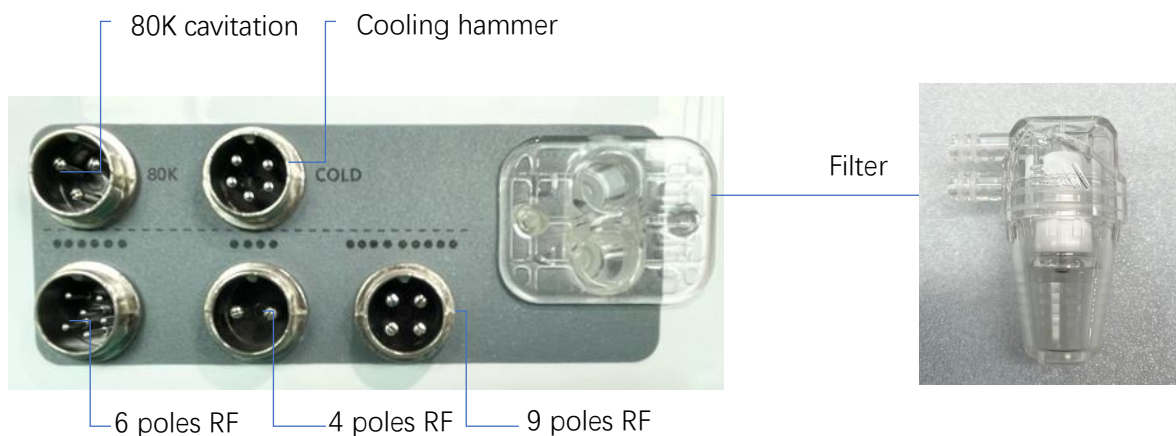

3) Connect each handle


As shown in the figure below, connect the handle wire to the corresponding interface on the back of the machine, and tighten the aviation plug.

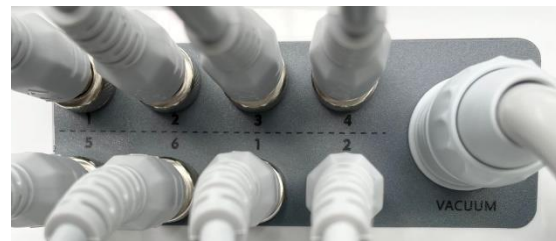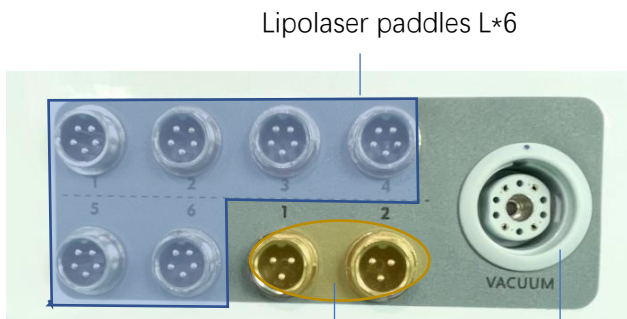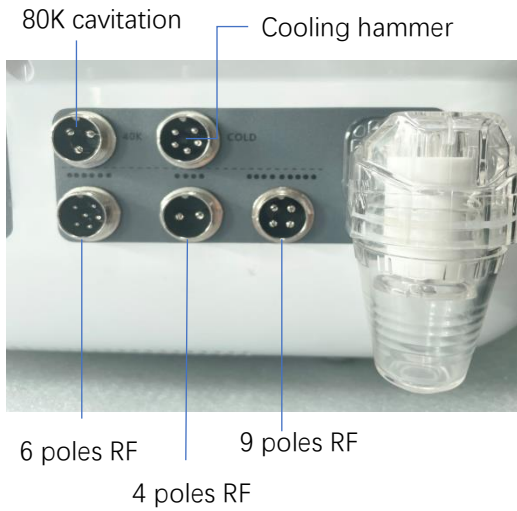

Lipolaser paddles S *2

6 poles RF +Vacuum

4. Operation Interface Instructions

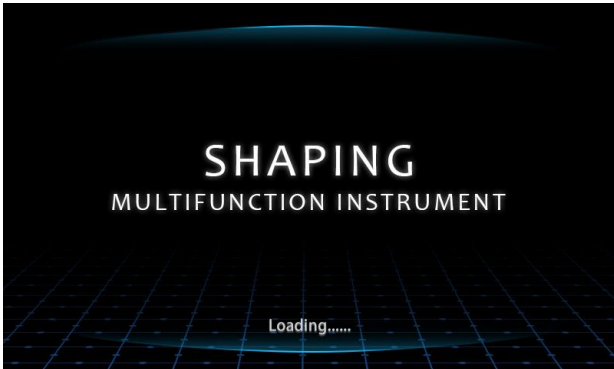

Startup interface



Silent cavitation

Continuous mode

Discontinuous mode

Audio cavitation

80K cavitation interface

【Click "Cavitation" to enter the following interface】



Cavitation Frequency

Step 2. Click AUTO to start searching for cavitation frequency

3. Click SAVE

Step 1. Choose Silent mode

Step 4. Choose audio mode, and repeat Step 2 and Step 3

80K cavitation frequency adjustment interface




RF + Vacuum interface



Discontinuous mode

Continuous mode

RF interface

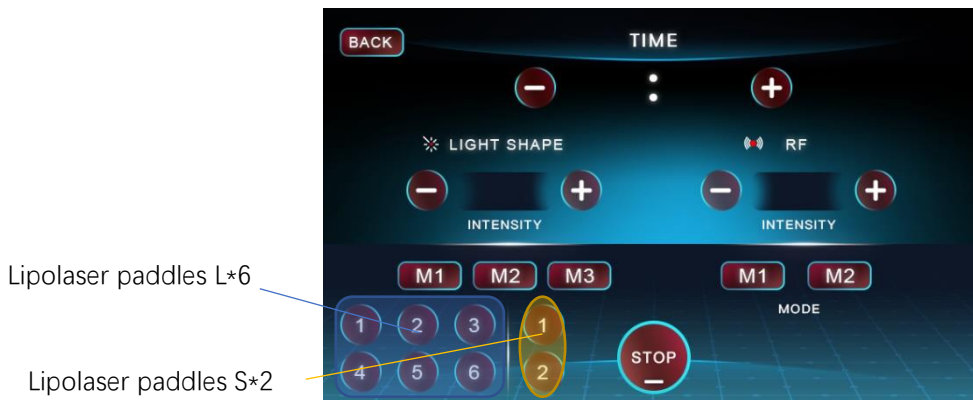

Lipolaser paddles L*6

Lipolaser paddles S*2

Lipolaser interface

Parameter

Product: Multipoles RF Vacuum Cavitation Body Slimming Machine

Input: AC 100V-240V 50/60Hz

Cavitation frequency: 80KHz

Vacuum Pressure: 650mmHg/0.8Bar

RF Frequency: 2MHz/4MHz

Laser Wave Length: 635-650nm

Cooling temperature: -1 -5°C